GUARDIAN®

Guardian® Platform Attachment

User Instructions & Warranty

Model 07702– Platform Attachment, Adult

Model 07703– Platform Attachment, Youth

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Consult your medical advisor for the proper use of this accessory. It is designed to be used on specific Guardian walkers that have two (2) attachment points that are 7/8" diameter. Use of this product on other than Guardian walkers may void warranty and may cause injury.

⚠ WARNING

For safety, these precautions must be taken when using this product .

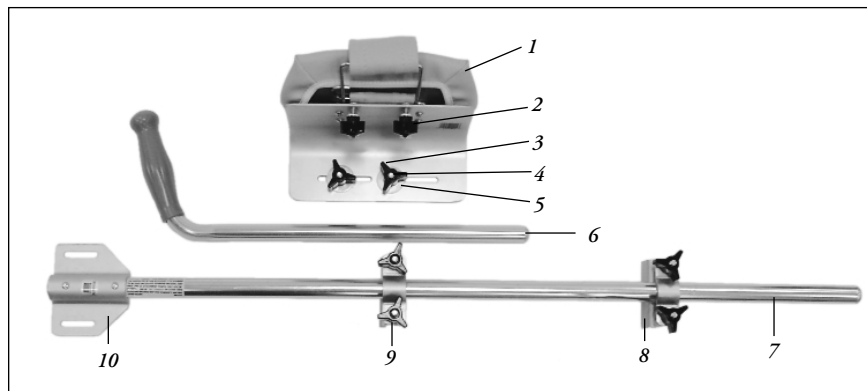
BEFORE EACH USE MAKE SURE:

1. You are correctly fitted.
2. Knobs are tightened and platform attachment is securely fitted to walker.
3. Grip, pad and velcro® are in good condition. Replace immediately when worn.
4. Hand grip is attached securely and does not slip.
5. Nothing is loose or uneven and there are no rattles or cracks in the metal, plastic, etc. or any other irregularities.
6. Product is used only with the manufacturer's parts and accessories.
7. User does not have any handicaps, disabilities or is on medication, etc. that a doctor, therapist, and/or seller would deem to interfere with the safe usage of the product.

Failure to follow the above instructions may result in serious injury.

IMPORTANT PARTS

The platform attachment consists of the following parts:

- | | |
|---------------------------|---------------------------|
| 1. Armrest | 6. Handle |
| 2. Armrest knobs | 7. Post |
| 3. Angle adjustment knobs | 8. Metal plates |
| 4. Bolts | 9. Post mounting knobs |
| 5. Washers | 10. Post mounting bracket |

ATTACHMENT TO WALKER

1. Remove angle adjustment knobs and washers from armrest. Assemble by inserting bolts through post mounting bracket. Add washers and tighten knobs (Figure 1).
2. Insert handle for right or left hand use so that the armrest is on the inside of the walker (Figure 2). Tighten knobs slightly.
3. With the post to the outside of the walker, center metal plates on top of upper and lower spreader bars (Figures 3 & 4). Tighten post mounting knobs so that platform attachment is securely in place.
4. Adjust handle to desired length and tighten armrest knobs securely.
5. Angle of armrest can be adjusted using the angle adjustment knobs (Figure 5). Consult your medical professional for proper adjustment.
6. Make sure all knobs are tightened and platform attachment is securely fitted to walker.

Figure 1

Figure 2

Figure 3

Figure 4

Figure 5

GRIP CLEANING INSTRUCTIONS

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

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LIFETIME LIMITED WARRANTY

Your Guardian® product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian® brand product or this warranty, please call us toll-free at 800.333.4000.

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