GUARDIAN®

Guardian® Sidestepper™

User Instructions & Warranty

Models 07770, 07771

Guardian® Sidestepper™

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Modelos 07770, 07771

IMPORTANT SAFEGUARDS

Important Information is highlighted by these terms:

WARNING– Important safety information for hazards that might cause serious injury.

CAUTION– Information for preventing damage to the product.

NOTE– Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

WARNING

For safety, these precautions must be taken when using this product. Before each use make sure:

1. Sidestepper is adjusted to correct height.
2. Detent buttons are fully engaged.
3. Tips are in good condition. Replace tips immediately when worn.
4. User weight does not exceed 250 lbs.

Use only as instructed by your healthcare provider.

WARNING

Always use extreme care when using this product. Avoid all potential safety hazards including slippery, uneven or soft surfaces and objects in path.

FAILURE TO FOLLOW THE ABOVE INSTRUCTIONS MAY RESULT IN SERIOUS INJURY.

SAVE THESE INSTRUCTIONS.

USER INSTRUCTIONS

Use only as instructed by your healthcare professional.

Opening and Closing

1. Open and close by moving handle located on lower half of Sidestepper.
2. In the open position, the handle should be horizontal and rest on the lower cross brace.
3. In the closed position, the handle should be vertical and the two halves of the frame will be folded against each other.
4. Be sure that the handle is in contact with the lower cross brace before using.

Height Adjustment

1. Adjust the Sidestepper so that when the arm is extended, the handle is at wrist height.
2. Depress spring button and slide leg up or down.
3. Engage spring button into hole that provides correct height.
4. Repeat for each leg, making sure Sidestepper sits level.

CLEANING

Grip Cleaning Instructions

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.

NOTE - Tips on this product contain dry natural rubber.

LIFETIME LIMITED WARRANTY

Your Guardian product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian brand product or this warranty, please call us toll-free at 800.333.4000.

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©2002, Sunrise Medical 01.02
Sunrise Medical Inc.