pump in style® advanced

breastpump
bomba de lactancia
tire-lait

- ✓ instructions
- ✓ instrucciones
- ✓ mode d'emploi

Table of Contents-English

Ordering parts	1
Breastpump parts list.....	1
Parts guide	2
Warranty	3
Health & safety information.....	4-8
Hygiene, cleaning, storing breastmilk	
Uses	5
Troubleshooting	9
Assembly instructions	
Pump In Style® Advanced breastpump	10-14
Pump In Style Advanced battery pack	15
Breastfeeding accessories (sold separately)	16

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Parts (See illustration on next page)

Item	Part No.
1. Ice pack (1)	#87092
2. Pump In Style® tubing	#8007212
3. 5 oz BPA-free breastmilk bottles (150 mL)	#8117012
4. Slow flow wide base nipple (1)	#8207060
5. Travel cap (1)	#2001686
6. Wide base nipple collar	#8107182
7. Solid container lids (4)	#8107183
8. PersonalFit™ breastshields (24 mm) (2)	#87073
9. PersonalFit™ connectors (2)	#87071
10. Spare membranes (6)	#87088
11. Spare valves & membranes (2)	#87089
12. 9 volt Pump In Style transformer* (1)	#9207010
13. 8-count battery pack* (1)	#67553
14. Diaphragm cap assembly	#6007105

Optional Accessories

9V Vehicle Lighter Adapter*	#67174
Breastmilk Bottle Set (8oz 3-pack)	#87132
Pump & Save™ Breastmilk Bags (20-pack)	#87033
Quick Clean™ Micro-Steam™ Bags	#87024

*For use with PNS Advanced versions sold after January 1, 2008

Ordering Parts

If your retailer doesn't carry the parts you need, in US and Canada, you can call 800-435-8316 (US and Canada), 815-363-1166 (US) or 905-608-8720 (Canada) and order with VISA, MasterCard, American Express, or Discover Card. In other countries please contact your local distributor. Visit www.medela.com for a distributor listing.

Pump In Style® Advanced Breastpump PARTS • PIEZAS • PIÈCES

**Pump In Style
Advanced
Shoulder Bag**

**Pump In Style
Advanced
Backpack**

**Pump In Style
Advanced
The Metro Bag™**

Medela Pump In Style® Advanced Breastpumps

This is a single user product. Use by more than one person may present a health risk and performance questions and voids the warranty.

This product is warranted by Medela, Inc. to the original retail purchaser to be free from defects in material and workmanship for the period of one year for pump mechanism (90 days for parts and accessories) from the date of purchase. In the event of a defect, Medela will repair or, at Medela's option, replace this product, without charge for such replacement, parts or labor. Purchaser shall bear all expense for returning this product to Medela. This warranty does not apply to any product used commercially or which has been subjected to misuse, abuse or alteration.

ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE WARRANTY OF MERCHANTABILITY, ARE LIMITED TO A DURATION OF 90 DAYS FROM DATE OF PURCHASE. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU. THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE.

Before you make a claim under this warranty, it may save time and expense to call Medela Customer Service (toll free) at 1-800-435-8316. You may also call this number for additional information concerning this warranty. If you wish to make a claim under this warranty, you must return this product to Medela with a return authorization number received from Medela Customer Service, prepaid, together with your dated bill of sale or other proof of purchase and a brief statement of the problem to the following address:

**Medela, Inc., P.O. Box 660, McHenry, IL 60051-0660
ATTENTION: SERVICE DEPARTMENT.
Call first for authorization number. Returns not accepted
without an authorization number.**

IMPORTANT SAFEGUARDS

This is a single user product. Use by more than one person may present a health risk and performance questions, and voids the warranty.

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS

DANGER: To reduce the risk of electrocution:

1. Always unplug electrical products immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product if it has fallen into water. Unplug immediately.

WARNING: To reduce the risk of burns, electrocution, fire or injury to persons:

1. A product should never be left unattended when plugged into an electric outlet.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use the product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
5. Keep the cord away from heated surfaces.
6. Do not use outdoors, or operate where aerosol (spray) products are being used or where oxygen is being administered.
7. Plug the transformer into Pump In Style Advanced product first and then into wall socket.
8. Only use the transformer that comes with the Pump In Style Advanced Breastpump.
9. Only use the Medela Pump In Style Advanced Battery Pak with the Medela Pump In Style Advanced Breastpumps.
10. This product is a personal care item and should not be shared between mothers.
11. Make sure the voltage of the transformer is compatible with the power source.
12. Pump only with the motor in an upright position.
13. Never use while sleeping or when drowsy.

SAVE THESE INSTRUCTIONS

Uses

- Collect breastmilk at work or during other absences from the baby.
- Maintain a milk supply if breastfeeding is temporarily interrupted.
- Relieve engorgement with single pumping. Single pumping with simultaneous massage is helpful to relieve engorgement.

Health & Safety Information

Important: This product is a single user item. Use by more than one user may pose a health risk and performance questions and voids the warranty.

Hygiene

Wash hands before touching breasts or containers and avoid touching the inside of containers or lids.

Cleaning

Follow these cleaning instructions unless you are told otherwise by your healthcare professional. Prior to cleaning, take apart all parts of your breastpump accessory kit. Especially be sure to remove the yellow valve from the breastshield and separate the white membrane from the valve prior to cleaning. See parts 8 - 11 on the illustration (page 2).

Prior to first use:

Disassemble all parts of your kit and breastpump that come in contact with the breast and milk (breastshields, connectors, valves, membranes, bottles, and caps). Sanitize in boiling water for 10 minutes. It is not necessary to sanitize tubing prior to first use.

After each use:

Disassemble and wash all parts that come in contact with the breast and milk in warm, soapy water (breastshields, connectors, valves, membranes, bottles, and caps). All parts are to be disassembled and washed separately for the pump to function efficiently.

- Rinse in clear water.
- Use a damp (not wet) cloth to wipe the bag and diaphragm cap of breastpump after each use. See part 14 on the illustration (page 2).

For Quicker Cleaning:

Medela offers Quick Clean™ Micro-Steam™ bags for daily kit disinfection. (Not recommended for use with SoftFit™ breastshields).

Tubing Care:

Inspect tubing after each pumping session for condensation and/or milk. It is not necessary to clean tubing after every use.

If condensation appears in tubing:

To eliminate condensation after you have completed pumping, continue running the pump with the tubing(s) attached for another 1-2 minutes or until dry.

If There Is An Overflow:

If milk accidentally gets into the tubings:

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing. Wash in soapy water, then rinse in cold clear water.
3. Shake water droplets out and hang to air-dry. For faster drying, attach tubings to pump and run pump 1-2 minutes or until dry, or pour a small amount of isopropyl alcohol through the tubings to dry. (Tubing can be cleaned using a Quick Clean Micro-Steam bag. Make sure tubing is fully immersed in water inside of bag).

If milk accidentally gets behind the diaphragm cap:

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing.
3. Snap off the diaphragm cap using thumb tab.
4. Be sure the yellow plug is not plugged into any ports prior to cleaning. Wash the diaphragm cap in soapy water, then rinse in cold, clear water.

5. Use a damp (not wet) cloth to wipe the diaphragm and bag. Do not immerse the motor unit or any of the bags in water.
6. When components are completely dry, reassemble breastpump and kit.

Storing Breastmilk

Check with your hospital for specific storage instructions.

When freezing, do not fill containers more than 3/4 full to allow space for expansion. Label containers with the date of pumping.

Freshly Expressed Breastmilk Storage Guidelines (For Healthy Term Babies)				
Room Temperature	Cooler with 3 Frozen Ice Packs	Refrigerator	Freezer	Thawed Breastmilk
4–6 hours at 66–78 °F (19–26 °C)	24 hours at 59 °F (15 °C)	3–8 days at 39 °F or lower (4 °C)	6–12 months 0–4 °F (-18–-20 °C)	use within 24 hrs

Reference: www.BreastmilkGuidelines.com

Important Note: If the temperature in the room, car or outdoors exceeds 77 °F / 25 °C, chill milk immediately to preserve freshness.

Storing breastmilk in the Pump In Style® Advanced cooler bag:

- The Pump In Style Advanced cooler bag requires one contoured ice pack (#87092, included) for cooling up to four bottles of breastmilk.

As soon as breastpumping is finished, place the breastmilk bottle or bag in the cooler bag with the ice pack. If the ice pack is frozen solid, it will safely cool your breastmilk for up to 12 hours. When you return home transfer the bottles to the refrigerator or freezer.

Defrosting

- Thaw milk overnight in the refrigerator, or hold the bottle under warm running water to quickly thaw. You can also place the sealed container in a bowl of warm water for 20 minutes to bring it to body temperature.
- Thawed milk is safe in the refrigerator for 24 hours. **Do not refreeze.**

Caution:

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- Never microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. Microwaving can also change the composition of breastmilk.
- If adding expressed breastmilk to container of already frozen breastmilk, make sure to add a lesser amount than the already frozen amount.

All Medela® breastmilk bottles and breastpump kits are 100% Bisphenol-A (BPA) Free.

Troubleshooting

If there is low or no suction

1. Make sure all connections of the kit and breastpump are secure.
2. If single pumping, make sure the port plug is placed into one of the single pumping ports.
3. Make sure the white membrane is laying flat against the yellow valve head. Check that the membrane is clean and not damaged. Refer to the assembly instructions.
4. Make sure the diaphragm cap or faceplate on the front of the breastpump fits evenly and securely.
5. Make sure the ends of the tubings are fit securely into the backs of the breastshields and into the ports on the diaphragm cap or faceplate.

Other concerns

6. Take care not to kink tubing(s). If milk is seen backing up into tubing, check that tubing is not kinked or pinched.
7. When traveling outside the U.S., use battery power only.
8. If you experience discomfort while pumping contact a breastfeeding specialist. You may need a larger size PersonalFit™ or SoftFit™ breastshield to reduce the possibility of nipple irritation or injury. Call 1-800-TELL YOU to locate a breastfeeding specialist in your area.
9. In the US, call Medela® at 800-435-8316 or 815-363-1166. In Canada, call Medela at 800-435-8316 or 905-608-7272. In other countries, contact your local distributor or the location where you purchased your pump.

Assembly Instructions - Pump In Style® Advanced Breastpumps

1. Securely attach breastshield to connector.

2. Snap membranes onto valves until membranes lie completely flat.

*Be sure to separate membranes & valve heads when cleaning and completely dry prior to assembly.

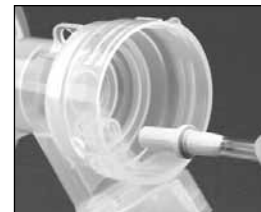
3. Push assembled valves & membranes onto breastshield connectors.

4. Screw bottles onto breastshield connectors.

5. Insert hard yellow adapter ends of tubing into openings in breastshield connectors.

6. Insert other ends of tubing into ports on diaphragm cap. To remove tubing, pull straight out of tubing ports; do not wiggle tubing or pull at an angle.

7. **If single pumping:** insert port cap into the unused port.

8. **If double pumping:** push port cap onto post.

9. Plug the metal fitting on end of transformer or Battery Pack cord into power jack on breastpump. Plug the 9 volt transformer into a standard electrical outlet.

10. Center the assembled breastshields over your nipples.

IMPORTANT:

Do not tilt or overfill containers when pumping.

IMPORTANT:

Take care not to kink tubing(s) while pumping.

11. To begin pumping, turn the dial clockwise from the off position. Pump In Style® Advanced will start in the stimulation phase. Adjust the speed/vacuum to your comfort level while pumping by turning the dial to a cycling speed most comfortable to you.

12. After two minutes, the stimulation phase will automatically change to the expression phase. If you let-down (your milk starts to flow) sooner than two minutes, you may pump more efficiently by pressing the let-down button at that time.

13. Once in the expression phase, set your Maximum Comfort Vacuum™. Maximum Comfort Vacuum is the highest vacuum you can use and still feel comfortable.

To find your **Maximum Comfort Vacuum**: increase speed/vacuum until pumping feels slightly uncomfortable (never painful), then decrease slightly.

14. If you need to return to the stimulation phase for any reason, simply press the let-down button. You may return back to the expression phase quickly by pressing the let-down button again.

**Research has shown that you will pump more efficiently - get more milk in less time - when pumping at Maximum Comfort Vacuum in the expression phase.*

Tip: Once you are finished pumping; unplug the tubing from the back of the breastshield prior to setting down the bottles. Allow the pump to continue to run while you safeguard your breastmilk. This will allow any condensation that may have formed in the tubing to dry.

Tip: Some moms prefer going back and forth between stimulation and expression to achieve second let-downs throughout a pumping session. You can do this by pressing the let-down button at any time.

Removing and securing the diaphragm cap:

Only remove diaphragm cap when cleaning is necessary. See page 6 for detailed cleaning information.

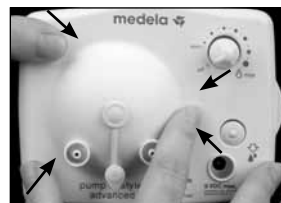
To remove the diaphragm cap:

1. Turn off pump, and unplug from power source.
2. Remove tubing (see instructions on page 11).
3. Snap off the diaphragm cap using the thumb tab (see left photo).

To secure the diaphragm cap:

Hold back of pump with one hand while pushing on the diaphragm cap until it snaps into place.

NOTE: The diaphragm cap must be completely snapped together at all connection points (see arrows) to ensure adequate vacuum and optimum pump performance.

Pump In Style® Advanced Battery Pack Instructions

The Pump In Style Advanced Battery Pack is intended for use with the Pump In Style Advanced breastpump (manufactured as of January 2008 or later) ONLY. The battery pack uses 8 AA alkaline batteries, or, for more consistent pump performance, use rechargeable batteries. Each set of batteries will provide approximately 2 hours of pumping time. It is recommended that batteries be replaced or recharged between the 10th and 12th pumping session.

Battery Insertion

Slide off battery covers. Place the NEGATIVE (-) end of the battery against the front of the spring. Push the battery in and down in one motion until the battery snaps into place. The POSITIVE (+) end of the battery should be near the positive sign on the battery pack. NOTE: If the battery is inserted in the opposite direction, the battery may not function correctly. Four batteries must be inserted on each side of the pack. Replace battery covers.

Breastfeeding Accessories

(sold separately)

Please check with your local retailer for any of these Medela® accessories to help make your breastfeeding experience more convenient, comfortable and enjoyable.

Pump & Save™ Breastmilk Bags

#87033 20-pack, #87034 50-pack

Pump & Save breastmilk bags are double-walled for safe and long-term breastmilk storage with a no-leak, easy-to-close zipper top and no-spill, stand-up bottom. Self-stick strap attaches directly to breastshield to pump directly into bag for fast, easy breastmilk storage.

Quick Clean™ Micro-Steam™ Bags

#87024

For breastpump and breastmilk feeding accessories, bottles, nipples, pacifiers and cups. Steam cleaning eliminates 99.9% of most common bacteria and germs. Requires just a microwave and water. Each bag can be used up to 20 times. Ideal for storing pump parts between pumping sessions.

Disposable Bra Pads

#89973 30-pack, #89974 60-pack

Compared to other disposable bra pads, Medela pads are proven to absorb more, so you use less. Super absorbent materials prevent leakage and maintain dryness. Soft layer wicks moisture away from breast for added comfort. Leak-resistant pocket and extra liner provide added protection. Contouring pleats ensure a smooth, feminine shape. Adhesive tape keeps pad in place and prevents shifting.

For a breastfeeding specialist or to locate Medela rental or retail locations, call the Breastfeeding National Network (BNN) at 1-800-TELL YOU; or visit the Medela website: www.medela.findlocation.com.

The Medela Customer Service Team can be reached at 1-800-435-8316.

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Printed in the USA. Medela, 2-Phase Expression, Lactina and Pump In Style are registered trademarks of Medela in the USA and a trademark in other countries. Quick Clean, Micro-Steam, Maximum Comfort Vacuum, Pump & Save, The Metro Bag, SoftFit and PersonalFit are trademarks of Medela, Inc. in the USA and in other countries. Applications for registration of Medela is pending in other countries. All rights reserved.

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Medela Canada, Inc.,